Minuet Watch PiperWatch

(EN) User Guide

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Tel: +44 (0)1977 661234
Fax: +44 (0)1977 660562
Email: enquiries@tunstall.co.uk
www.tunstallgroup.com

Tunstall

Tunstall Group Ltd
Whitley Lodge
Whitley Bridge
Yorkshire
DN14 0HR

Minuet Watch
PiperWatch

Part No. D6707228A

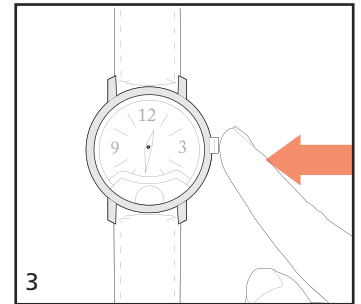
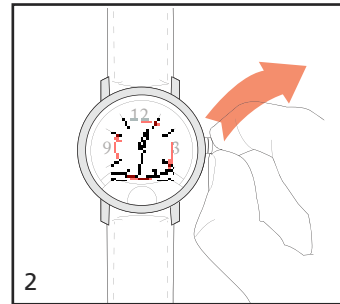
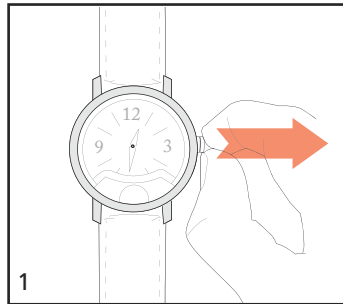
(EN) Setting the time

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



(EN) Wearing options - changing from watch to pendant and vice versa

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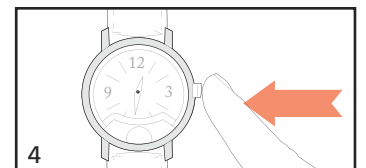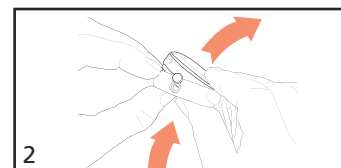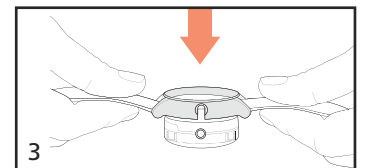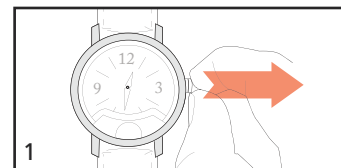
(EN) Programming - place the home unit in programming mode and press the alarm button on the watch, then make a test call.

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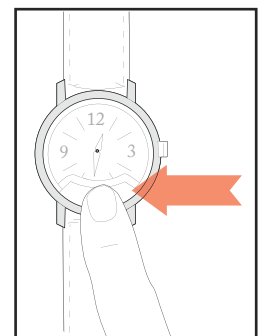
(EN) Making an emergency call - press the alarm button on the watch. The red LED at the top of the watch face should light up. If the LED flashes when the button is pressed the radio transmitter battery is low. The alarm button should be tested regularly.

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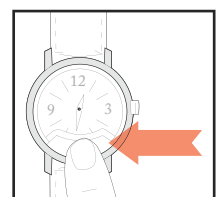(EN) Battery change - the radio transmitter battery can be changed by suitably skilled individual by removing the small screws on the back of the watch. The watch battery itself cannot be changed.

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(EN) Cleaning - use only clear water and soap

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(EN) Technical Data

Radio Frequency:	869.2125MHz
Radio Range:	Up to 50m (in building), 200m (line of sight)
Battery Type:	Radio Transmitter: CR2032
Battery Life:	Watch: 5 years (non replaceable) Radio Transmitter: 3 years (replaceable)
Dimensions:	46 x 38 x 13 mm (L x W x D)
Weight:	24g
Operating temperature:	-10°C to +55°C
Watch accuracy:	-10/+20 sec / month (at 25°C)
Waterproof:	IP67
CE:	Compliant
Standards:	EN 50134-2

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(EN) Declaration of conformity

We, Tunstall declare that this telecare equipment is in compliance with the essential requirements and other relevant provisions of the R&TTE Directive 1999/5/EC.

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