



# Care Assist

## What is it?

The CareAssist is a portable telecare alarm that can receive telecare alerts and display not only the type of sensor activated but also either the location of the sensor and/or the name of the person the sensor has been assigned to. This means that onsite carers can be quickly made aware of any incidents allowing them to provide a high level of care whilst maximising their efficiency. It will also help to reduce the impact on informal carers' lifestyles enabling them to lead more independent lives.

## How does it work?

Telecare sensors are pre-assigned to the CareAssist and allocated to either a location within a building e.g. communal lounge or to a specific resident e.g. Mrs Smith Flat 1. When the telecare sensor is activated, the CareAssist will vibrate, beep and light up its bright screen to alert the carer to the incident allowing them to view what sensor activated the alarm, which room/resident it is allocated to and the time and date of the alarm enabling them to take speedy and appropriate action.

## Who is it for?

CareAssist is designed to support local carers to deliver high quality, non-intrusive care. It is extremely easy to use, portable device that provides carers working or living onsite with a means to receive instant alerts from Tunstall's range of telecare sensors.

As a result, onsite carers can be quickly made aware of any incidents and provides a very cost effective telecare solution whilst avoiding the need for a telephone line or monitoring centre service which is often not required when full time (24 hour) care is being provided.

With a radio range of up to 200 metres (line of sight), it is particularly suited to small specialist dementia care, intermediate care, learning disability, residential & care home, and respite care facilities.

It can also provide significant benefits to private individuals and their informal carers who may prefer not to be connected to a 24 hour monitoring centre service.

## Features and Benefits

- **Compatible with Tunstall telecare range**  
Supporting up to 86 telecare sensors
- **Easy to use - with clear luminous 2.4"**  
OLED display
- **Fully portable** - small pocket sized design and battery operated
- **Multiple alert modes** - the unit can alert the carer using audible alerts, screen message and a vibration mode. The audible alerts and vibration mode can be turned off if necessary.

## Why Tunstall?

We focus on using the latest digital and mobile technology to enable people to feel safe, secure and independent, giving them the freedom to live the life they choose. Our products combine se-cure digital connectivity and mobile platforms.

## We help you provide...

- **Intelligent, unobtrusive, person centred care.**
- **Personalised, proactive and predictive services to improve quality of life.**
- **Integrated health, housing and social care.**

For more information please visit:

[tunstall.co.uk](http://tunstall.co.uk)

## Specification

### Technical

**Weight:** 148g

**Dimensions:** 128 x 73 x 22mm (L x W x D)

**Radio frequency:** 869.2125MHz

**Radio range:** Up to 2 hours (requires initial 24 hour charge)

**Battery Backup Time:** Up to 12 hours

**Battery Charge Time:** 139 locations (inc 64 custom locations)

**Programmable Sensors:** Up to 86

**Number of locations/users:** Last 256 events

**Memory capacity: Warranty:** 2 years

**Accessories supplied:** Belt clip, USB charger, Antenna, User Guide

### Standards

**CE:** Compliant

**Product:**  
CareAssist

**Part Number**  
68100/01

The following accessories are also available:

#### Accessory:

PC download lead (RS232):

**Part Number**

D6705015A

RS232 to USB converter:

S8502004

#### Replacement Accessories:

Battery

**Part Number**

S1004037

Belt clip

D6702098A

Antenna

S2005018

Micro USB charger

XD6706011A

#### Pager System Accessories

Pager

**Part Number**

i9000/69A

Pager Transmitter\*

i9000/61A

Pager Transmitter Connection lead (Connects CareAssist to the Transmitter)

D6705015A

Tunstall Healthcare (UK) Ltd is a member of the Tunstall Group.

Contact us on: **t: 01977 661234** | **e: enquiries@tunstall.com** | **w: tunstall.co.uk** | **@TunstallHealth**

Our policy of continual development means that product specifications and appearance may change without notice. Tunstall does not accept any responsibility for any errors and omissions contained within this document. © 2019 Tunstall Group Ltd.

Version 2019\_v0